


Attention Modules

The course of therapy can be individualized to each patient's specific needs based on module difficulty and their current ability. Most therapists start with attention therapy using RehaCom module "Attention and Concentration."

Attention and Concentration

Start here →

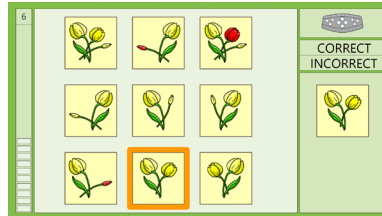

Start each new patient at Level 1. If after a few rounds, the level seems too easy, press the ESC button and change "Current Level of Difficulty" to level 6; proceed with the training.

If Attention and Concentration is too difficult, then continue with Alertness Modules.

Easier ↓

Harder ↓


If Attention and Concentration is not challenging enough, then continue with Spatial Attention or Divided Attention Modules.

Alertness



Alertness Module

Spatial Attention

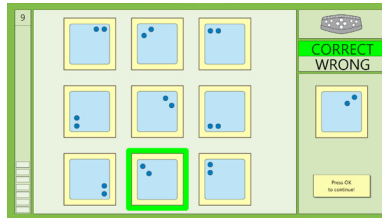


Two-dimensional Operations


Divided Attention



Divided Attention

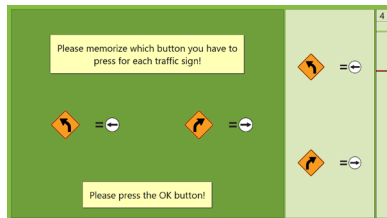

Reaction Behavior

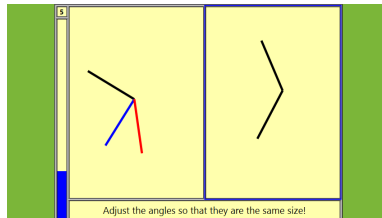


Spatial Operations



Divided Attention 2



Vigilance

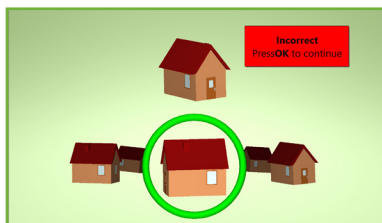

Spatial Operations 3D



Plan a Vacation
from Executive Functions domain

888.783.6363 |  | PearsonClinical.com

Copyright © 2017 Pearson Education. All rights reserved. Always Learning, Pearson, Psi design, and PsychCorp are trademarks, in the U.S. and/or other countries, of Pearson Education, Inc. or its affiliates. RehaCom is a registered trademark of HASOMED GmbH. Pearson Clinical Assessment, a business unit of NCS Pearson, Inc., is the authorized distributor of RehaCom within the United States. CLINA15776-30920 ML 04/17